

Grip Ribbon – Non-slip mat

Manual



Checked 12.6.26

care
care

www.en.carecare.fi

Non-slip mat can be used in various applications such as on tables, chairs, and floors. The product is used to increase friction between contacting surfaces. Works e.g. under a trays and cutting boards, on the chair under a padding and under the carpet. Material is durable and maintains friction properties also when wet. Do not apply excessive force when stretching the mesh structure.

Can be washed in a washing machine at 60 degrees Celsius with mild detergent, rinsing is recommended. Use at room temperature.

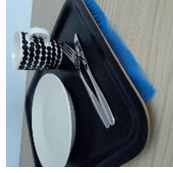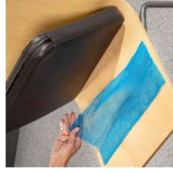

Technical information

Non-slip mat.

Quantity: 1 pc. 25 cm x 15 cm (L x W).

Material: SEBS. Free from latex and silicone. Made of synthetic rubber.

Recycling: General waste.

Medical device. Date of manufacture indicated on the barcode label.

Any serious incident should be reported to CareCare Ltd. and to the Finnish Medicines

Agency (Fimea) using the User Incident Report Form.

Basic UDI-DI 643005903GR020WC

Manufacturer:

CareCare Ltd. FI-MF-000017782

Kirkkokatu1a A6

FI-80110 Joensuu

www.en.carecare.fi

sales@carecare.fi

Sales +358 50 511 2779

Production +358 50 524 7559



SEBS



Made in
Finland



Kitkanauha - Liukestealusta

care
care



www.carecare.fi

Käyttöohje

Tarkastettu 12.6.26

Liukeste, jota voi käyttää monipuolisesti mm. pöydillä, tuolilla ja lattialla. Käytetään lisäämään kitkaa kosketuksessa olevien pintojen välillä. Toimii esim. tarjottimen ja leikkuulustan alla, tuolilla pehmusteen alla ja maton alla. Materiaali kestää hyvin kosteutta ja säilyttää kitkaominaisuutensa myös kosteana. Älä venytä verkkorakennetta voimakkaasti. Ei sisällä lateksia eikä silikonia.

Voidaan pestä 60°C konepesussa miedolla pesuaineella, suositellaan huuhtelua. Käytetään huonelämpötilassa.

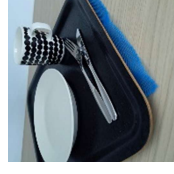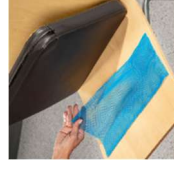

Tekniset tiedot

Liukeste.

Koko: 1 kpl. 25 cm x 15 cm (P x L).

Materiaali: SEBS. Ei sisällä lateksia eikä silikonia. Valmistettu synteettisestä kumista.

Kierrätys: Sekajäte.

Lääkinnällinen laite, valmistus pvm. viivakooditarrassa.

Vaaratilanteista pyydetään ilmoittamaan CareCare Oy:lle ja Fimeaan Käyttäjän

Vaaratilanneilmoituslomakkeella.

Basic UDI-DI 643005903GR020WC

Valmistaja:

CareCare Oy FI-MF-000017782

Kirkkokatu1a A6

FI-80110 Joensuu

www.carecare.fi

sales@carecare.fi

Myynti +358 50 511 2779

Tuotanto +358 50 524 7559



SEBS



Valmistettu
Suomessa

