

# Grip Ribbon – Opening Aid

care  
care



www.en.carecare.fi

## Manual

Checked 12.6.26

Grip Ribbon is used to increase friction between contact surfaces when grip strength is weakened. You can fold the mesh structure twice for a better grip. The grip strength required to open the jars can be reduced further by placing another piece of Grip Ribbon under the jar or under the support hand. Also functions as an aid for opening medicine packages. Do not apply excessive force when stretching the mesh structure.

Can be machine washed in a 60-degree Celsius with mild detergent, rinsing is recommended. The product is used at room temperature.

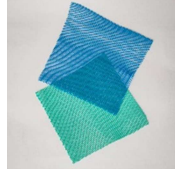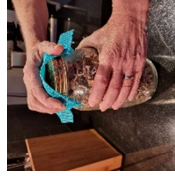

# Kitkanauha - Avaaja-Apu

care  
care




www.carecare.fi

## Käyttöohje

Tarkastettu 12.6.26

Käytetään lisäämään kitkaa kosketuksessa olevien pintojen välillä, kun otevoima on heikentynyt. Voit taittaa verkkorakenteen kaksiin kerroin paremman otteen saamiseksi. Purkkien tai pullojen avaamiseen tarvittavaa puristusvoimaa voidaan pienentää asettamalla toinen pala Kitkanauhaa avattavan purkin tai pullon alle tai tukikäden alle. Toimii myös lääkepakkausten avaamisen apuvälineenä. Verkkorakennetta ei kannata venyttää.

Voidaan pestä 60°C konepesussa miedolla pesuaineella, suositellaan huuhtelua. Käytetään huonelämpötilassa.

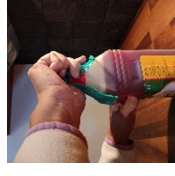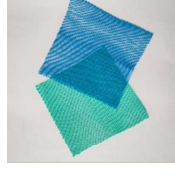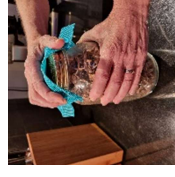

### Technical information

Opening aid for jars and bottle caps.

**Quantity:** Two pieces, each measuring 15 × 15 cm in size.

**Material:** SEBS. Free from latex and silicone. Made of synthetic rubber.

**Recycling:** General waste.

Medical device. Date of manufacture indicated on the barcode label.

Any serious incident should be reported to CareCare Ltd. and to the Finnish Medicines

Agency (Fimea) using the User Incident Report Form.

Basic UDI-DI 643005903GR020WC

**Manufacturer:**

CareCare Ltd. FI-MF-000017782

Kirkkokatu1a A6

FI-80110 Joensuu

www.carecare.fi

sales@carecare.fi

**Sales:** +358 50 511 2779

**Production:** +358 50 524 7559

### Tekniset tiedot

Apuia purkkien ja korkkien avaukseen.

**Koko:** Kaksi 15 x 15 cm kokoista palaa.

**Materiaali:** SEBS. Ei sisällä lateksia eikä silikonia. Valmistettu synteettisestä kumista.

**Kierrätys:** Sekajäte.

Lääkinnällinen laite, valmistus pvm. viivakooditarrassa.

Vaaratilanteista pyydetään ilmoittamaan CareCare Oy:lle ja Fimeaan Käyttäjän

Vaaratilanneilmoituslomakkeella.

Basic UDI-DI 643005903GR020WC

**Valmistaja:**

CareCare Oy FI-MF-000017782

Kirkkokatu1a A6

FI-80110 Joensuu

www.carecare.fi

sales@carecare.fi

**Myynti** +358 50 511 2779

**Tuotanto** +358 50 524 7559

